



## Resolution of MS Update Issue on Spirotrac V Print

(Note: Administration rights are required to run this file.)

1. Log on as a user with Admin rights on the required PC.
2. Ensure Spirotrac V is closed.
3. Download the patch file and run it (Browse to downloaded file, default location is the Downloads folder).

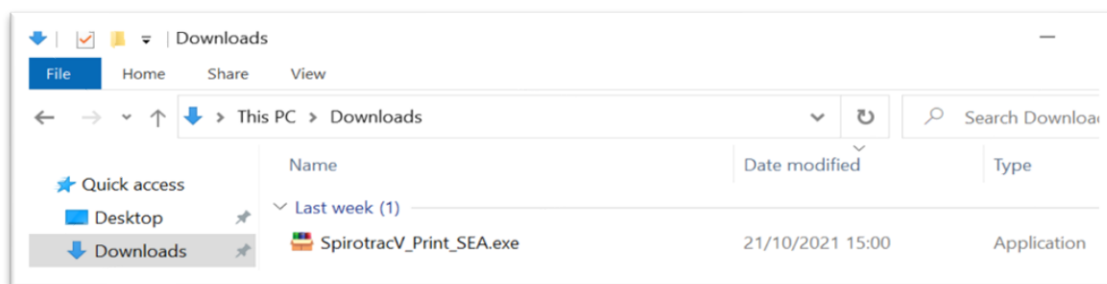

4. The following screen should appear:

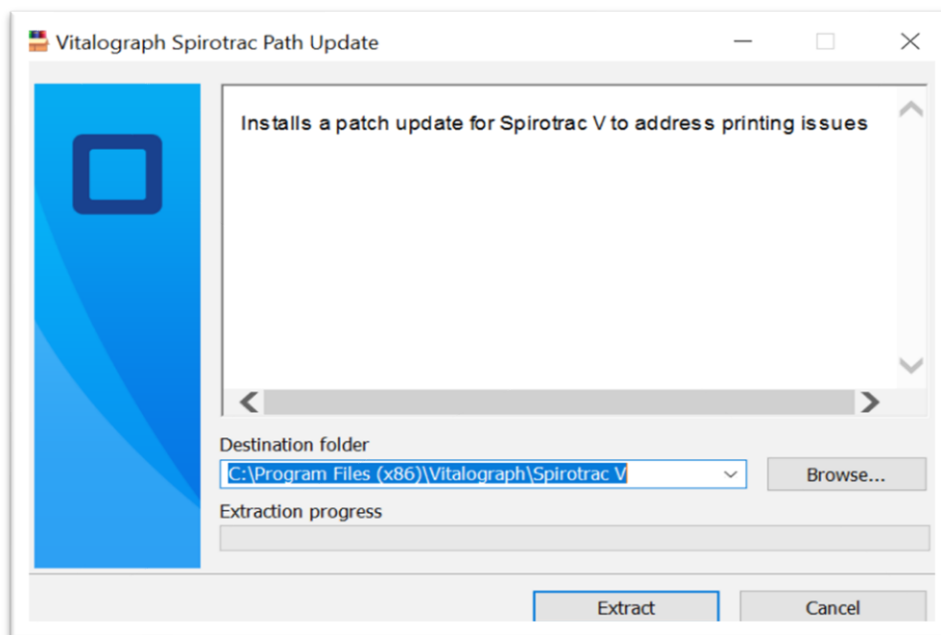


Note: The Destination folder must be the folder location where Spirotrac V is installed. The default location is C:\Program Files (x86)\Vitalograph\Spirotrac V.

To select a different location, select Browse and browse to Spirotrac V installation location.

5. Select Extract

The following prompt will be reported on screen:

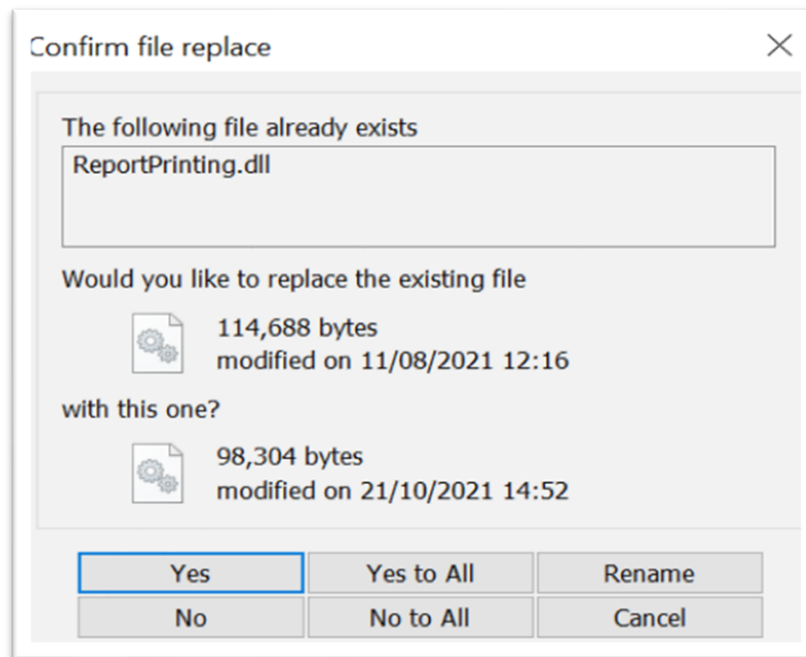

6. Select Yes. On completion, log on to Spirotrac and test.