

In2itive Spirometer

A powerful and versatile respiratory diagnostics solution with wireless bi-directional EMR connectivity, enabling detailed, point-of-care spirometry testing wherever it is needed.



A Global Leader in **Respiratory Diagnostics**

Point-of-care spirometry – an essential element of respiratory health management

Spirometry is the term given to the basic lung function tests that measure the air that is expired and inspired. It is performed to detect the presence or absence of lung disease, quantify lung impairment, monitor the effects of occupational/environmental exposures and determine the effects of medications.¹

Point-of-care testing allows the performance of a test in the immediate vicinity of a patient, providing rapid results. Simple and non-invasive, point-of-care spirometry testing is suitable for multiple healthcare settings, outside the conventional laboratory environment.

Conditions that benefit from point-of-care spirometry testing include:

[COPD](#) | [Respiratory Tract Infections](#) | [Lung Cancer](#) | [Asthma](#) | [Many other respiratory-related conditions](#)

Powerful spirometry in the palm of your hand for accurate and connected testing, wherever it is needed.

The In2itive™ delivers detailed point-of-care spirometry that is compliant with ATS/ERS 2019 standards for accuracy and test quality.

Ideal for use throughout secondary care, the In2itive has an easy-to-navigate interface and can store more than 10,000 tests. Its robust, detachable flowhead is suited to a variety of testing scenarios. A large touch screen enables the clinician to actively coach the patient during testing and to give real-time feedback on effort and test quality.

Bi-directional EMR connectivity continuously synchronises respiratory assessment needs and spirometry results, to maximise the efficiency of healthcare decisions.



Multiple testing options

Test Modules	Description	Parameters*	Onscreen Graphs
Forced Vital Capacity	The total volume of air that can be exhaled during a maximal forced expiration effort.	FVC, FEV₁, FEF₂₅₋₇₅, PEF , etc	<ul style="list-style-type: none"> • Flow / Volume / Time • Volume / Time
Vital Capacity	The total volume of air that can be exhaled during a normal expiration effort.	VC, IC, IVC, TV	<ul style="list-style-type: none"> • Bar Chart • Volume / Time
Bronchodilator Responsiveness Testing	Determines the degree of improvement of airflow in response to bronchodilator administration as measured by changes in FEV ₁ and FVC.	VC, FVC, FEV₁, FEF₂₅₋₇₅, PEF , etc	<ul style="list-style-type: none"> • Pre-Post Flow / Volume • Pre-Post Volume

Powerful, versatile, connected.

Exceptional Test Accuracy

- Features robust V-Core™ flowhead measuring technology which offers spirometry testing that is accurate, repeatable, easy to use, and safe.
- Accuracy meets or exceeds the performance standards ATS/ERS 2019, ISO 23747:2015 & ISO 26782:2009.

Seamless Connectivity

- Bi-directional data exchange to EMR is achieved through Vitalograph Connect, a PC application which is provided as standard.
- Compatible with HL7, GDT and several regional specific protocols.
- Multiple connectivity options, including wireless and USB.
- PDF reporting software.

Hygienically Efficient

- Low running costs: no disposable sensors, turbines, or flow tubes.
- Single-use Vitalograph Bacterial Viral Filters (BVF™) with validated cross-contamination efficiency >99.999% protect device, patient, and operator.

Easily Interpreted Results


- Fast evaluation of results with %Predicted comparison, LLN and Z scores.
- Latest GLI predicted sets.
- Real-time flow/volume onscreen graphics.
- Instant test quality feedback.

Testing Versatility

- Long-life rechargeable battery.
- 10,000 test capacity.
- Connected or detached flowhead.
- Lightweight and handheld.



Be free to help your patients perform spirometry

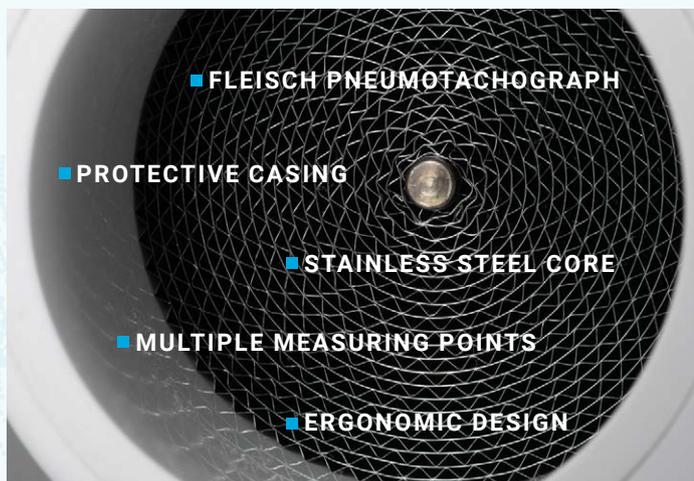

The In2itive has an integrated **V-Core™** flowhead, allowing you to focus on helping your patients achieve their best spirometry test, confident that you are capturing accurate and reproducible results.

V-Core Flowhead Technology is:

- Accurate
- Repeatable
- Robust
- Safe
- Comfortable

2019
ATS/ERS
Spirometry Technical
Standards
Compliance

Technical Specifications

Product: Vitalograph In2itive

Model: 2120

Flow detection principle: Fleisch type pneumotachograph

Volume accuracy: +/- 2.5% ATS/ERS 2019, ISO

26782:2009

Flow accuracy: ±10% or ±10L/min of the reading (ISO

23747:2015)

Performance standards: ATS/ERS 2019, ISO 23747:2015

& ISO 26782:2009

Safety standards: EN 60601-1:2006 +A1:2013 +A2:2021

QA/GMP standards: EN ISO 13485, FDA 21 CFR 820,

CMDR SOR/98-282, JPAL & MDSAP

Dimensions: 186mm x 81mm x 48mm

Weight: 300g

Communications: USB 2.0 & Wi-Fi

Power Supply: 3.7V, 2Ahr Lithium Polymer battery

Parameters measured: EVC, IVC, FRC, VC, FVC, FIVC/
FVC, FEV_{0.5'}, FEV_{0.5'}/FVC, FEV_{0.75'}, FEV_{0.75'}/FVC, FEV_{1'}, FEV_{1'}R,
FEV_{1'}/VC, FEV_{1'}/EVC, FEV_{1'}/FVC, FEV_{1'}/FIVC, FEV_{1'}/FEV_{6'},
FEV_{1'}/PEF, FEV_{3'}, FEV_{3'}/VC, FEV_{3'}/FVC, FEV_{6'}, PEF L/s, PEF
L/min, FEV_{25'}, FEF_{50'}, FEF_{75'}, FEF0.2-1.2, FEF_{25-75'}, FIV_{1'}, FIV_{1'}/
FVC, FIV_{1'}/FIVC, PIF L/s, PIF L/min, FIF_{25'}, FIF_{50'}, FIF_{75'}, FEF_{50'},
FIF_{50'}, MVVind, FMFT, FET, FRC, TV, RV, TLC, IRV, ERV, IC,
Rind, ELA, tPEF, Text, BEV, BEV/FVC, EOTV, tRISE, tHES,
FIVC, FEV_{1'}/FVC, FEF_{25-75'}/FVC, FEF_{75-85'}, FIF_{50'}-FEF_{50'}

Ordering Information

Product Model Number	Customer Order Number	Product Description
2120	83002	Vitalograph In2itive

Optional Extras / Consumables

Product Model Number	Customer Order Number	Product Description	Product Box Size
2040	36020	3L Precision syringe	1
2820	28501	Eco BVF for Office Spirometers	100
	28572	Eco BVF for Office Spirometers + Disposable Nose Clip	80
	28554	Eco BVF with Plastic Bite Lip	75



Free 5-Year warranty included with product registration.

Contact us

Our customer service team are available for telephone support on sales, technical queries or information requests.

USA & Canada

Vitalograph Inc.

Toll Free: 1-800 255 6626
Tel: +1 913 730 3200
Fax: +1 913 730 3232
contact@vitalograph.com
13310 West 99th Street,
Lenexa, Kansas, 66215, USA

Ireland

Vitalograph (Ireland) Ltd

Tel: +353 65 686 4100
Fax: +353 65 682 9289
sales@vitalograph.ie
Gort Road Business Park,
Ennis, Co Clare, V95 HFT4,
Ireland

United Kingdom

Vitalograph Ltd, UK

Tel: +44 1280 827110
Fax: +44 1280 823302
sales@vitalograph.co.uk
Maids Moreton,
Buckingham, MK18 1SW,
England

Germany

Vitalograph GmbH

Tel: +49 40 547391-0
Fax: +49 40 547391-40
info@vitalograph.de
Rellinger Straße 64a
D-20257 Hamburg,
Germany

All other countries

Vitalograph Ltd, International

Tel: +44 1280 827120
Fax: +44 1280 823302
sales@vitalograph.co.uk
Maids Moreton,
Buckingham, MK18 1SW,
England



Vitalograph, Vitalograph®, In2itive™, V-Core™ and BVF™ are trademarks or registered trademarks of Vitalograph Ltd.