

Vitalograph copd-6 range

COPD screening monitors

Vitalograph copd-6

Case selection for spirometry and monitoring diagnosed COPD sufferers

- ❖ For the early detection of COPD - quickly, simply & accurately
- ❖ Identifies those at risk of COPD at the pre-symptomatic stage to allow early medical intervention and facilitate better clinical outcomes
- ❖ Screens out those whose FEV₁ is normal, and who therefore do not have COPD, without the risk of false COPD negatives
- ❖ Facilitates 'case selection' so that spirometry resources can be focused on those most likely to be diagnosed with COPD
- ❖ Monitors COPD patients using their 'number', the obstructive index – FEV₁ as a percent of predicted
- ❖ Displays FEV₁, FEV₆, ratio and % predicted, obstructive index, COPD classification and lung age
- ❖ Built-in quality of blow indicator on a large, easy to read display
- ❖ Easy to clean flowhead
- ❖ Can be used with hygienic SafeTway[®] mouthpieces or a BVF[™]
- ❖ Displays the GOLD COPD classification (stage I – IV) to help recognise the need for a change in the patient's management plan
- ❖ Requires only minimal instruction for use by non-respiratory specialists

The copd-6[™] comes complete with:

- ❖ 3 SafeTway mouthpieces
- ❖ Carry pouch
- ❖ Guide for the healthcare professional
- ❖ 2 AAA Batteries

The copd-6 is very easy to use, simply:

- ❖ Enter age, height and gender
- ❖ Blow for 6 seconds (device beeps after 6 seconds)
- ❖ Blow 3 times
- ❖ Press 'Enter' to view results
- ❖ Transmit report to PDF from USB variant

Vitalograph is a world leading provider of outstanding quality cardio-respiratory diagnostic devices, clinical trial services and medical equipment servicing. With a pioneering heritage of excellence spanning half a century Vitalograph continues to make valuable contributions to effective medical care and enhanced quality of life.



Vitalograph copd-6 usb

Compatible with Vitalograph Reports Software

- ❖ Single page record for screening or monitoring
- ❖ Various report options
- ❖ Optional subject ID and name fields (weight field optional)
- ❖ Ability to add comments to the report
- ❖ Age, height, gender, device ID, time/date fields automatically populated
- ❖ Predicted values, 3 test results, best test, % predicted and lung age automatically tabulated
- ❖ Obstructive index, COPD (GOLD) Classification indicator
- ❖ BMI calculated
- ❖ Blow quality indicator
- ❖ Interpretation of results
- ❖ Can automatically name the PDF file
- ❖ Ability to customise report headers
- ❖ Ability to add comments to the report



Additional Report Information

The following details are not stored on the copd-6 device. If you would like any of these details to be displayed on the generated report, please enter them now. Otherwise, they shall not appear on the report.

Demographics

Last Name: _____ Initial: _____ First Name: _____

Subject ID: _____ Weight: (lbs) _____

Comments: _____

Unit Measure
 Imperial & US
 Metric

Edit Custom Header Continue >>

The copd-6 usb comes complete with:

- ❖ Reports software & API documents CD
- ❖ USB cable
- ❖ 4 SafeTway mouthpieces and disposable noseclips
- ❖ Carry pouch
- ❖ Guide for the healthcare professional
- ❖ 2 AAA batteries



Vitalograph
COPD Screening Report

This is a Customer Header for Company XYZ, Dept ABC

Subject Information						
Subject Name:	John Doe	Subject ID:	564565			
Gender:	Male	Age:	37			
Height:	175 cm	Regression Set:	Temp			
Weight:	70 kg	BMI:	22.9			

Test Information						
Device ID:	1234567890	Report Date:	09-Jul-2008 13:28			
Device Firmware Ref:	04108	Device Software Ref:	V1.06			
Number of Blows:	10	Number of Good Blows:	5			
Best FEV1 within:	0.04 L	Best FEV6 within:	0.2 L			

Results						
Parameter	Normal Predicted	Test 1	Test 2	Test 3	Best Test	% Pred
FEV1 (L)	3.96	2.25	2.21	2.16	2.25	57
FEV6 (L)	4.78	2.48	2.28	2.16	2.48	52
FEV1/FEV6 (ratio)	0.82	0.90	0.96	0.99	1.00	110

Obstructive Index		COPD (GOLD) Classification	
Normal	≥ 80%	Normal	FEV1/FEV6 ≥ 0.70
Mild	< 80%	Stage I	FEV1/FEV6 < 0.70 & FEV1 ≥ 80 % Pred.
Moderate	< 50%	Stage II	FEV1/FEV6 < 0.70 & FEV1 < 80 % Pred.
Severe	< 30%	Stage III	FEV1/FEV6 < 0.70 & FEV1 < 50 % Pred.
		Stage IV	FEV1/FEV6 < 0.70 & FEV1 < 30 % Pred.

FEV1 Measured/FEV1 Predicted

Interpretation

Lung Age: 89 years

Obstructive Index: 57%

Interpretation: Normal, not COPD
Possible Restriction

Comments

Test Report

Signature: _____ Date: _____

Vitalograph Reports - Software Reference: 65030 V1.20.0023 Page 1 of 1

4000 Respiratory Monitors

Variant	PEF	FEV ₁	FEV ₆	FEV ₁ Ratio	FEF _{25-75%}	FEV _{0.5/0.75}	Lung Age	Memory	Personal	GOLD Best	PDF Reports	SPV Reports
asma-1™ standard	●	●	○	○	○	○	○	600	●	○	○	○
asma-1 child version	●	●	○	○	○	●	○	600	●	○	○	○
asma-1 usb/s/bt	●	●	○	○	○	○	○	600	●	○	●	●
copd-6™ standard	○	●	●	●	○	○	●	○	○	●	○	○
copd-6 usb/s/bt	○	●	●	●	○	○	●	○	○	●	●	○
lung monitor standard	○	●	●	●	○	○	○	200	●	○	○	○
lung monitor usb/s/bt	○	●	●	●	●	○	○	200	●	○	●	○
lung age standard	○	●	○	○	○	○	●	○	○	○	○	○

● Available ○ Not available

usb - USB

bt - Bluetooth

s - Serial

Technical Specification:

Product: Respiratory Monitor

Model Number: 4000

Parameters Displayed: FEV₁/ FEV₆/ ratio and % predicted

Quality of Blow Indicator: Yes

Obstructive Index: Mild, Moderate, Severe (FEV₁% predicted)

COPD Classification: Normal (not COPD); Stage I, II, III & IV (FEV₁% predicted)

Lung Age Indicator: Yes

Accuracy: Better than +/-3%

Range: 0 - 9.99 L BTPS

Sensor: Stator rotor

Flow Impedance: Better than 0.15 kPa/L/s at 14 L/s

Power Supply: AAA batteries

Display: Custom liquid crystal display

Size: 113 x 63 x 48mm

Weight: 55g

Performance Standards: ISO 26782:2009; ISO 23747:2007; ATS//ERS 2005

Operating Temperature: 17 - 37°C

Safety Standards: IEC 60601-1:2005

Medical Safety Standard: Medical Devices 93/42/EEC (as amended)

Designed & Manufactured Under: ISO 13485:2003, FDA 21CFR820

Ordering Info:

40200 4000 Respiratory Monitor copd-6

40450 4000 Respiratory Monitor copd-6 usb

40850 4000 Respiratory Monitor copd-6 s

40350 4000 Respiratory Monitor copd-6 bt

20242 2024 SafeTway mouthpieces (200)

20980 2024 SafeTway mini mouthpieces (50)

28350 2820 BVF Bacterial Viral Filters (50)

20303 2030 Noseclips disposable (200)

36020 2040 Precision Syringe 3 Litre

Vitalograph®

Your respiratory partner

UK Sales
Vitalograph Ltd.
Maids Moreton, Buckingham,
MK18 1SW, England
Phone: (01280) 827110
Fax: (01280) 823302
e-mail: sales@vitalograph.co.uk
www.vitalograph.co.uk

Export Sales
Vitalograph Ltd.
Maids Moreton, Buckingham,
MK18 1SW, England
Phone: +44 1280 827120
Fax: +44 1280 823302
e-mail: sales@vitalograph.co.uk
www.vitalograph.eu

Vitalograph GmbH
Rellinger Straße 64a,
D-20257 Hamburg,
Germany
Phone: (040) 547391-0
Fax: (040) 547391-40
e-mail: info@vitalograph.de
www.vitalograph.de

Vitalograph Inc.
13310 West 99th Street,
Lenexa, Kansas, 66215,
USA
Toll Free: 800 255 6626
Phone: (913) 730 3200
Fax: (913) 730 3232
e-mail: vitcs@vitalograph.com
www.vitalograph.com

Vitalograph (Ireland) Ltd.
Gort Road Business Park,
Ennis, Co. Clare,
Ireland
Phone: (065) 6864100
Fax: (065) 6829289
e-mail: sales@vitalograph.ie
www.vitalograph.ie

Vitalograph Ltd.
Unit 230, Corporation Park,
11 On Lai Street,
Shatin,
Hong Kong
Phone: +852 2117 2678
Fax: +852 2117 2679
e-mail: sales@vitalograph.hk
www.vitalograph.hk

www.vitalograph.co.uk

Vitalograph®, asma-1™, copd-6™, SafeTway® and BVF™ are trademarks or registered trademarks of Vitalograph Ltd.
Bluetooth is a registered trademark of the Bluetooth SIG.

CE
0086

PRINT REF: 14498_5