**CE** Peak Flow Meter  
 Ref. No. 43900  
 0086  
 EN ISO 23747  
 Model 4300 S-10991691-4390091  
 Manufacturer: Vitalograph, Limerick, Ireland

*my PEF*  
**Peak Flow Meter**  
 Your respiratory partner  
**Vitalograph**

the difference is clear

**ENG Use, Cleaning & Regulatory Notices**

**Setting The Action Zones**

The zone markers may be set with a stylus by the healthcare professional to give the personal boundaries for implementing the asthma action plan.

**How To Use The Peak Flow Meter**

- Slide the pointer to the bottom of the scale
- Hold the device horizontally in front of your mouth
- Inhale as deeply as possible
- Seal your lips firmly around the mouthpiece
- Blow out for a second or longer as HARD and as FAST as possible
- Take care that the tongue or teeth do not obstruct the airflow. A 'spitting' action or coughing will give false readings
- Take note of the reading and slide the pointer back to the bottom of the scale
- Repeat this three times and record the highest reading
- Take action according to your personal asthma management plan

**Cleaning & Disinfecting**

Use of an ordinary alcohol wipe (IPA 70-90%) is recommended with special attention to the mouthpiece area. Visually contaminated devices should be cleaned and disinfected prior to use.

**Single Person Home Use**

Clean at least weekly. Replace after 3 years.

**Clinic Use** (requires mouthpiece adaptor)

Between each subject: Clean, disinfect, and fit a new disposable Safeway or BVF mouthpiece. It is recommended that peak flow meters used in clinic be replaced every year or after 650 patients whichever is earlier.

**Warnings & Advisory Notices**

*Note: Please read all this information before using this Peak Flow Meter.*

- This Vitalograph device is intended to help you assess how well your asthma is under control, by measuring your peak flow which can then be monitored over time
- Never attempt to dismantle the unit. This can cause faulty Peak Expiratory Flow scores
- If used at home symptoms must take precedence over device measurements
- If you think that the device is not reading correctly, advise your healthcare professional immediately
- If the device is used for longer than it's specified life, the accuracy of the device may deteriorate
- A damaged Peak Flow Meter should always be replaced
- Store in a clean dry place

**Warranty**

Your Vitalograph device is guaranteed for one year\*.

\* Excepting accidental / transit damage or inappropriate use of the device.

**CE Notice**

Marking by the symbol CE indicates compliance of this device to the Medical Devices Directive of the European Community. Such marking is indicative that the Vitalograph device meets or exceeds the referenced technical standards.

**FDA Notice**

Caution: Federal Law restricts this device to sale by, or on the order of a physician.

**Declaration Of Conformity**

**Product:** Vitalograph Model 4300  
**Vitalograph hereby ensures and declares that the above product associated with this user manual, is designed and manufactured in accordance with the following QMS regulations and standards:**

- European Medical Devices Directive 93/42/EEC, as amended. This device is classified as 1m per Annex IX of the MDD also meets the provisions of the Essential Requirements; Annex I, via compliance with Annex II of the Medical Devices Directive as per Article 11, section 3a, excluding point 4 of Annex II;
  - Canadian Medical Device Regulation (CMDR SOR/98-282);
  - FDA Quality System Regulation (QSR) 21 CFR 820;
  - EN ISO 13485:2003. Medical devices. Quality management systems. Requirements for regulatory purposes
- Certifying Body(For 93/42/EEC and CMDR): British Standards Institute, Certificate Nos. CE 00772, CE 85553, MD 82182, FM 83550.

**Technical Specifications**

**Product:** Peak Flow Meter  
**Model Number:** 4300  
**Material:** Recyclable ABS plastic  
**Accuracy:** ± 10 L/min or ± 10% of the reading  
**Repeatability:** ± 10 L/min or ± 5% of the reading  
**Highest resistance to flow:** 0.00377kPa/L/min @ 720 L/min  
**Measurement Range:** 50-800 L/min BTPS

**Operating Temperature Range:** ATS/ERS Limits: 17-37°C. Design Limits: 10-40°C  
**Temperature:** 0-50°C Relative Humidity: 10%-95%

**Peak Flow Meter performance standards:**  
 EN ISO 23747:2009; ATS/ERS:2005; AS/NZS4237;  
**Frequency:** Profile A/B difference less than 15 L/min/12% (Annex B; EN ISO 23747:2009)  
**Medical Safety Standard:** Medical Devices Directive 93/42/EEC1993 as amended  
**Designed & Manufactured Under:** ISO 13485:2003, FDA 21CFR820, CMDR & JPAL

Signed on behalf of Vitalograph (Ireland) Ltd.

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Gezeichnet im Namen von Vitalograph (Ireland) Ltd.