

Errata

Installing the *Vitalograph® Pneumotrac* USB Drivers

The installation of the Vitalograph Pneumotrac USB driver is a two-step process. Please be sure to follow both Step 1 and Step 2 below, otherwise the driver will not be installed completely and the Vitalograph USB Pneumotrac device will not work.

Step 1

1. Make sure the Spirotrac CD is inserted in the CD/DVD drive of the PC. If not, insert it now. (Note: When the CD is inserted, the installation wizard will start. You can cancel this)
2. Connect one end of the USB cable into an available USB port on the PC
3. Connect to the other end of the USB cable into the Vitalograph Pneumotrac device
4. The PC will detect that new hardware has been connected and “New Hardware” installation wizard shall be displayed
5. You may be asked if Windows can connect to Windows Update to search for the hardware driver. Choose “No, not this time”.
6. You will be asked where to install your hardware from and choose “install the software automatically”
7. You may be presented with an information dialog in relation to Windows Logo checks. Choose “Continue Anyway” on this screen.
8. The first USB Pneumotrac driver will be installed at this point and you will be presented with a “Finish” screen. Click this button and follow Step 2.

Step 2

9. When you have clicked the “Finish” button, you will once again be presented with the same screen stating that “New Hardware” has been detected. This is because the USB Pneumotrac Device requires two hardware drivers.
10. Follow steps 4 through 8 again. When you complete these steps, your USB Pneumotrac will be ready to use.

Important Note:

Having completed the installation, you need to know what port number has been assigned by your PC to the USB Pneumotrac Device.

To do this follows these steps:

- Go to Windows “Control Panel”.
- Click on the “System” icon
- Click on the “Hardware” tab
- Click on the “Device Manager” button
- Expand the “Ports (COM and LPT)” tree node
 - Look for the “Vitalograph USB Pneumotrac (COM x)” where x represents a number. This is the Port number to which the USB Pneumotrac is connected.
 - Make sure this is the Port number selected for the Pneumotrac device in the Spirotrac settings.